

External Positive Control

Be sure your test kit is working properly.



Why To Use

It is recommended to apply the positive control in the following situations.:

- Once for each new lot or shipment of kits received.
- Once for each new operator.
- As deemed additionally necessary to conform with your internal quality control procedures, with local, state and/or federal regulations, or accrediting groups.

How To Use

External Control Preparation

1. Open the positive control pouch and place.
2. Add 100 μ l (~3 drops) extraction buffer solution to the positive control tube and mix well.

External Control Procedure

1. Place the test device at a flat surface and apply 100 μ l positive control solution to the sample well on the test cassette.
2. Read the test result in 15 minutes.

